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Description

The ADF2 respiratory protection mask protects the wearer from potentially harmful solid and/or non-volatile liquid particles.


The ADF2 mask is classified as FFP2 and complies with the provisions of the European directives:

- Regulation on personal protective equipment (EU 2016/425)
- EN 149:2001+ A1:2009 (Respiratory protective devices - Filtering half masks to protect against particles)

Instructions for use

Before use, wash your hands and check that the face mask is not damaged.

1. Wash your hands.
2. Hold the mask in your hands with the back facing you.
3. Unfold the outer sides. Both elastic bands must be outside the bowl shape on the upper side.
4. Hold the face mask in one hand with the opening towards the face. The aluminium nose bridge must be on top.
5. Open the mask to form a bowl shape.
6. While holding the face mask at chin height, pull both elastic straps, one by one, behind your head.
7. Lay the upper elastic strap over the crown of your head and the lower elastic strap under your ears. Make sure the elastic straps are lying flat along their whole length and adjust the position if necessary for a comfortable fit. The top of the mask covers the nose, the bottom covers the chin.
8. Use both your hands to shape the aluminium nose bridge around your nose and cheeks. Ensure a correct seal with the nose bridge.
9. Carry out the fit test correctly before entering the workplace.
10. Wash your hands.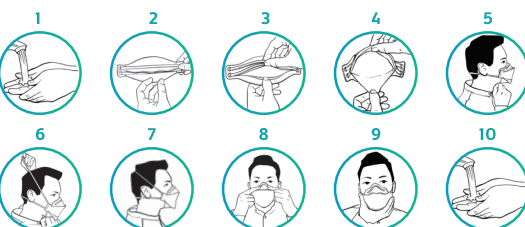

Fit test


Use both hands to check for any leaks on and around the face mask during the following exercise.

1. Take a deep breath.
2. If air leaks out along the edges of the mask, adjust the position of the elastic straps for a better fit.
3. If air leaks out along your nose, adjust the nose bridge for a better fit.
4. Repeat the previous steps until no more air leaks out.

If you do not succeed in wearing the face mask correctly, do not enter the contaminated area.

Removal instructions

1. Wash your hands.
2. Do not touch the face mask. Gently pull the lower elastic strap over your head.
3. Then also pull the upper elastic strap over your head and remove the face mask. Dispose of the face mask according to your facility's policy.
4. Wash your hands.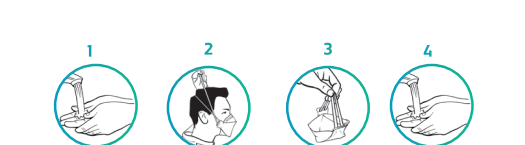

Warnings

- The mask is suitable for single use only and must not be used for more than one shift.
- Use the mask strictly as instructed. Non-compliance may lead to exposure to contaminants and the associated health damage.
- Selection of the correct type of masks and pre-use training are essential to protect the user from potentially harmful solid and/or non-volatile liquid particles.
- Do not use the mask in combination with beards or other facial hair that may reduce the edge sealing between the face and the mask.
- Use the mask during the entire exposure time to contaminants.
- The mask does not protect against potentially harmful volatile gases or vapours.
- Do not use the mask in rooms with an oxygen content of less than 19.5%.
- Ne pas utiliser dans des zones à atmosphère explosive.
- The masks do not contain latex.
- Before use, check that the date of use is earlier than or equal to the expiry date indicated on the mask.
- Only remove the film in which the mask is packed immediately before use to prevent contamination.
- If the mask gets damaged, the breathing resistance becomes too high or if the mask has been contaminated with infected material such as blood, you must leave the area where potentially hazardous substances are present and replace the mask.
- These products do not eliminate the risk of developing a disease or infection.
- Do not modify the mask design, do not clean it for reuse and do not repair the mask if it is damaged.

Storage conditions and packaging

The mask must be stored in a clean dry environment at a temperature between 0°C to +30°C and at a maximum humidity of < 80%. The product has an expiry date 3 years after its production date.

Explanation of symbols

NR = Non-reusable. Only for use during one shift.

	Batch number
	Expiry date
	Temperature range
	Air humidity
	Instructions for use
	Manufacturer
	Alerts
	For single use only
	Manufacturing date and time

Destruction

Destroy the mask in accordance with local regulations

Contact details of the manufacturer and notified body
The masks are manufactured by Medimundi BV, Visbeekstraat 22, 2300 Turnhout, Belgium. The declaration of conformity can be found at www.medimundi.be.
Notified Body Centexbel Belgium (number 0493), Technologiepark 70, BE 9052 Gent) carried out the EU type-approval and issued type-approval certificate.